



Veracyte Afirma Genomic Testing

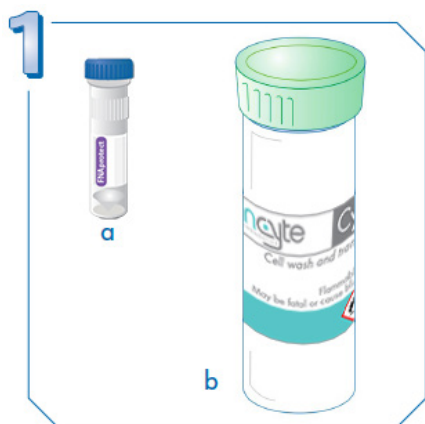

Remove small FNAProtect tube (a) from kit. FNA collection for cytology can be submitted, expressed into a green top vial (b), if desired (collect three passes as usual and include prepared slides)

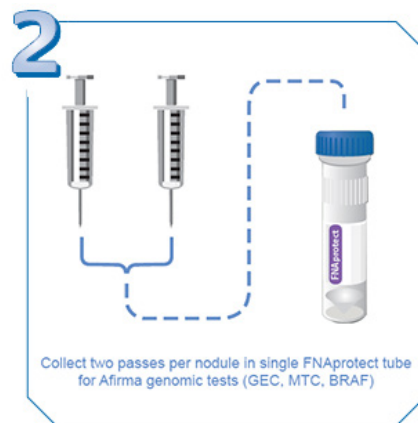

For the Afirma collection, express 2 complete passes (4th & 5th pass if FNA cytology is also submitted) into the vial.



Secure screw cap on vial and invert 2-3 times. Write patient's full name on requisition sticker and affix to vial(s).



Place the Afirma FNAProtect tube inside the provided padded envelope.



Place the padded envelope with optional green top vial and prepared slides into the plastic specimen bag.



Order the Afirma Genomic Sequencing Classifier. Include complete patient information, ICD-10 codess, and target lesion size and location. Place the completed requisition into the specimen.